# ARTROMOT®-E2 compact

**USA** Operation Manual

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# Contents

<b>Device description of the ARTROMOT®-E2 compact</b>	
Front side	2
Back side	40
<b>Figures of the ARTROMOT®-E2 compact</b>	41
<b>1. How to use the motion device</b>	4
1.1 Areas of Applications	4
1.2 The aims of therapy	4
1.3 Indications	4
<b>2. Description of the ARTROMOT®-E2 compact</b>	4
<b>3. Safety information</b>	5
<b>4. Adjusting the device</b>	6
4.1 Connecting the <b>ARTROMOT®-E2 compact</b>	6
4.2 Adjusting the <b>ARTROMOT®-E2 compact</b> to the patient	7
<b>5. Programming the ARTROMOT®-E2 compact</b>	8
5.1 Programming steps	8
5.2 Information on treatment values	9
5.3 Programming the special functions	10
<b>6. Conversion</b>	12
<b>7. Shipment/set up and transport</b>	13
<b>8. Care</b>	13
<b>9. Specifications</b>	15
<b>10. Contacts</b>	18
<b>11. Technical service</b>	19
11.1 Technical hotline	19
11.2 Replacing fuses	19
11.3 Shipment	18
<b>12. Spare parts</b>	19
<b>13. Declaration of conformity</b>	21

# 1. How to use the motion device

## 1.1 Areas of Application

The **ARTROMOT®-E2 compact** is a motorized motion device used for **Continuous Passive Motion (CPM)** of the elbow joint.

Its use is an important contribution to medical-therapeutic treatment both in clinics and doctors' offices and in rental services.

## 1.2 The aims of therapy

Motion therapy with the **ARTROMOT®-E2 compact** is particularly effective in the prevention of immobilization damage, the early restoration of pain-free joint mobility and the promotion of speedy healing with good functional results.

Other objectives of the therapy include:

- improvement joint metabolism
- prevention of joint stiffness
- promotion of the rebuilding/healing of cartilage areas and damaged ligaments
- increasing the speed of haematoma resorption
- improvement lymph and blood circulation
- prevention of thrombosis and embolism

## 1.3 Indications

The motion device is indicated in the treatment of most injuries, postoperative states, and diseases of the elbow joint, for example:

- joint distortions and contusions
- arthrotomy and arthroscopy procedures in combination with synovectomy, arthrolysis, or other intraarticular measures
- joint mobilization during narcosis
- fractures and pseudoarthroses that are stable during exercise after operation
- operations to replace muscle
- arthroplasties, including endoprosthesis implantation
- corrective osteotomy

The **ARTROMOT®-E2 compact** should **not be used** for:

- acute inflammations changes in joints, unless expressly prescribed by a doctor
- spastic lameness
- unstable osteosynthesis

## 2. Description of the ARTROMOT®-E2 compact

The **ARTROMOT®-E2 compact** facilitates passive movements of the right and left elbow joints. It can be converted in just a few steps.

The motor-driven motion device enables movements in the elbow joints in terms of extension/flexion – 5-0-140 degrees and Pronation/Supination 90-0-90 degrees.

The **ARTROMOT®-E2 compact** features the following:

- anatomically correct adjustments
- physiological movement processes
- the widest possible range of motion
- hand-held programming unit for precise adjustments of all treatment values
- Patient chip-card to store the programmed values
- easy-to-transport design

### Explanation of the functional elements of ARTROMOT E2 compact

**Note: See page 2 and 21**

- 1 Screw for height adjustment
- 2 Screw for elbow angle settings
- 3 Eccentric lever for lower arm length adjustment
- 4 Lower arm support with hand rest

- 5 Button for upper arm support swivel
- 6 Screw for lower arm element swivel
- 7 Screw for right/left positioning
- 8 Handheld programming unit
- 9 Storage for handheld programming unit
- 10 Patient chipcard
- 11 Main plug
- 12 Main switch
- 13 Fuse
- 14 Connector for mobility element
- 15 Adjusting screw
- 16 Strap for lower arm support
- 17 Motor A
- 18 Motor B
- 19 Upper arm support
- 20 Strap for upper arm support
- 21 Roller brake
- 22 Transport rollers
- 23 Stop bolt for height adjustment of upper arm
- 24 Connector for handheld programming unit

### Explanation of symbols

	Alternating Current
	Safety conductor connection
	Applied part type B
	Main switch OFF
	Main switch ON
	Caution consult accompanying documents

## 3. Safety information

### CAUTION!

**This section must be read before starting the device**

- keep in mind that the plugs for the mobility element can only be connected in one direction
- make sure that the mobility element's connector is always locked on using the safety hinge
- the **ARTROMOT®-E2 compact** may only be operated by **authorized personnel**
- only the original patient chip card may be used
- make sure that the patient sits in an anatomically **correct position**
- always use a four leg chair without armrest when operating the **ARTROMOT-E2 compact**

### **Adjustments of ARTROMOT®-E2 compact:**

1. Elevation adjustment
2. Rest angle adjustment
3. Lower arm length adjustment
4. Axis of motor B adjustment

**see page 22**

- movements must be **free of pain and irritation**
- the patient must be fully conscious when receiving instruction and when using the motion device
- in individual cases, the doctor or therapist must decide whether the motion device can be used with the patient
- when treating adipose, particularly large and particularly small patients, make sure to avoid abrasion and pressure points

- the maximum continuous load on the arm bearing elements is 9 kg/20 lbs
- if there is any doubt about the correct setting and/or movement ranges, the therapy should be stopped immediately. Please check the CPM device and notify ORMED Customer Service if necessary (see chapter 11)

### **CAUTION!**

**Before beginning with the treatment, a test run of several motion cycles must be carried out without the patient, and then with the patient.**

- the hand-held programming unit should be explained to the patient and must always be located within the patient's reach, in order to enable interrupting the therapy.

- Make sure that the rating of your power supply network matches the voltage and frequency ratings on the ID plate.
- The **ARTROMOT®-E2 compact** should only be connected to correctly installed safety contact sockets
- Repair and maintenance work should only be carried out by authorized persons as otherwise all warranty and liability on the manufacturer's part shall lapse

- Perform regular checks on all components for possible damage or loose connections
- Damaged or worn parts should be replaced immediately by authorized special personnel using original spare parts
- Before carrying out any cleaning and maintenance work, the main plug must be disconnected from the socket.
- No liquid should be allowed to reach the inside of the housing of the electrical components when carrying out work on the device

### **CAUTION!**

**The device may not be used to transport people**

### **CAUTION!**

**The maximum load of the seat is 150 kg / 330 lbs.**

### **CAUTION!**

**The maximum load of the arm bearing elements is 9 kg/20 lbs.**

## **4. Adjusting the ARTROMOT®-E2 compact**

### **4.1 Connecting the ARTROMOT®-E2 compact**

**Note: to illustrate the steps described, see page 22**

- First connect the **main plug (11)** to a safety socket and switch on the **main switch (12)**

- Insert the original patient chip card (10) in the handheld programming unit (8)

### **CAUTION!**

**Once the button START has been pressed, the device will automatically move into the programmed motion of range. Once the range of motion has been reached, the device will stop.**

To begin the treatment, please program and restart the device.

### **CAUTION!**

**Please check: Does the rotation axis of the motion device match the rotation axis of the elbow joint? Does the rotating axis of Motor A run horizontally through the elbow joint? Does the rotating axis of Motor B run vertically through the elbow joint?**

## **4.2 Adjusting the ARTROMOT®-E2 compact to the patient**

- Activate the programming mode by briefly pressing the Extension/Flexion and STOP keys simultaneously or by holding the STOP button down 5 sec.
- Move the motion device into a position (Extension/Flexion, Pronation/Supination) that is comfortable for the patient
- Perform the adjustments from step 1 to 4

### **① Elevation Adjustment (Fig. A)**

Hold the mobility element at Motor A (17). Open the screw (1) and set the height. Make sure that the upper arm is positioned exactly on the upper arm support (19). The patient should have a straight and comfortable position.

Lock the screw (1). Make sure that the axis of Motor A is in concordance with the elbow joint.

If necessary, the precise adjustment in vertical direction can be done by turning the screw (2).

### **② Rest angle adjustment (Fig.B)**

Hold the bar of the motion element. Open the screw (2) and set the desired angle. Lock the screw.

### **CAUTION!**

**Hold Motor B before adjusting.**

### **③ Lower arm length adjustment (Fig. C)**

Open the eccentric lever (3) and set the desired length.

The patient fingers' should not touch Motor B. Lock the eccentric lever.

### **④ Axis of Motor B adjustment (Fig. D)**

The standard positioning for this adjustment is generally "0" on the scale, which is situated close to Motor B.

According to deformities of the lower arm, it can be changed.

Anatomically correct setting

## 5. Programming the ARTROMOT®-E2 compact

### CAUTION!

Make sure that movements are free of pain for the patient. Before beginning with the treatment, a test run of several motion cycles must be carried out without the patient, and then with the patient


The following **treatment values** can be set using the **handheld programming unit**:

- Extension/Flexion
- Pronation/Supination
- Pause for Extension/Pronation
- Pause for Flexion/Supination
- Motor A/B On/Off
- Speed
- Timer

In addition, **special functions** can be programmed or called (see chapter 5.3).

**Programming is only possible when the patient's chip card is inserted in the handheld programming unit.**

## 5.1 Programming steps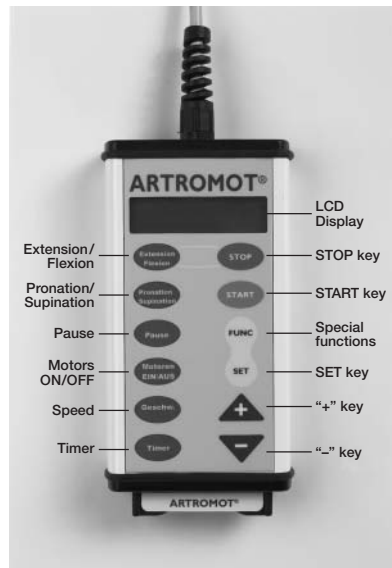

1. Press **Extension/Flexion** and **STOP** simultaneously or press **STOP** for 5 sec. to enter the programming mode.

2. The treatment values can now be selected in succession by pressing the parameter keys. **If the keys have a dual assignment, press the key twice to set the second treatment value.**
3. Use the +/- keys to change the value:
  - hold down +/- to move quickly through the values
  - the desired motion dimensions have to have at least a difference of 5°
 If this is not the case, the display notice "DIFF" will appear.
4. Continue programming (points 2 and 3) until all of the desired values of range of motion have been entered
5. Press **STOP** to save all of the previous values.
6. Press the parameter keys with the device stopped to display all of the currently saved values.
7. Press **START**. The device checks the settings (display "Move to ROM") and then stops.
8. Press **START** again to begin treatment.

### **Important!**

When using an already programmed patient chip card, press **START** after the card has been entered without the patient.

The motion device automatically moves into the programmed motion range and stops.

Press **START** again to begin with the treatment.

### **Important!**

In emergency cases, in order to interrupt the treatment, press any key on the hand-held programming unit and the device will stop.

## 5.2 Information on treatment values

### Maximum settings for motion range

<b>Extension</b>	-5°
<b>Flexion</b>	140°
<b>Pronation</b>	-90°
<b>Supination</b>	90°

### Pauses

- The pauses occur at the limit positions of motors A and B and can be set separately
- Settable values for pauses: settable in seconds from 0 to 59 seconds, and then in minutes from 1 to 60 min.

### Motors ON/OFF

Motors A and B can be used together or separately. By pressing the +/- key, you can switch ON or OFF the motor A (Extension/Flexion) or the motor B (Pronation/Supination).

Make sure that at least one motor is always switched on. If this is not the case, the notice "MOTOR A/B: OFF!" will appear.

### Speed

The minimum speed is  
1% = 2,3°/min.

The maximum speed is  
100% = 230°/min.

By pressing the **SPEED** key, you can then set up the speed with the +/- key from 1 to 100%.

### Timer

- The values that can be set for the timer (therapy time) with the +/- key: **continuous operation of 1–59 min** and then **1–24 h** in 30 min steps.
- The device **automatically** switches into the middle of the set values when

the therapy time has elapsed

- In **continuous operation**, the device can be switched off by pushing the **STOP** key

### 5.3 Programming the special functions of the ARTROMOT®-E2 compact

The possible special functions are:

- New Patient
- Warm-up
- Therapy duration
- Reverse on load
- Asynchronous
- Display lighting
- Language
- Service menu

#### Programming the special functions:

1. Switch to the programming mode (ch. 5.1)
2. Press **FUNC**.
3. Select special functions using the +/- keys
4. Confirm by pressing **SET**
5. Terminate programming by pushing the **STOP** key
6. The device moves into its factory settings and stops automatically
7. Press **START** again to begin the treatment

#### New patient

Please activate this special function to reset the device to its factory settings. All of the values programmed are deleted.

The function "New Patient" contains the following settings:

- Extension: 25°
- Flexion: 35°
- Pronation: -5°
- Supination: 5°
- Pause for Extension/Pronation: 0 sec.

- Pause for Flexion/Supination: 0 sec.
- Motors A/B: ON
- Speed: 100 %
- Timer: Continuous operation
- All special functions are deactivated
- Motors run synchronously
- Reverse: 25

#### Warm Up

The **ARTROMOT®-E2 compact** offers a warm-up program as a special function which starts at the middle position of programmed ROM.

Once the warm-up program has been activated, the device begins to move between Extension and Flexion, as well as Pronation and Supination, to reach within 15 motions cycles the programmed ROM. When this phase is over, the programmed range of motion begins.

#### Therapy duration

Under menu item "Therapy Duration", the duration of the whole therapy can be called for each patient.

1. Select the function "**Therapy Duration**" using the +/- keys. The therapy duration will be displayed
2. Press the **SET** key for 5 sec. to delete the therapy duration

#### Reverse-on-load

The device switches into the opposite motor direction when the patient's resistance exceeds the set values.

#### Adjustable levels for reversal: 1-25

Level 1 = easy / level 25 = hard

1. Select "reverse" using the +/- keys
2. Press **SET**
3. Use the +/- keys to set the desired level
4. Activate the setting by pressing **SET** again
5. Use the +/- keys in order to select the next function or terminate programming

## CAUTION!

Load reverse is exclusively a safety measure for cramps, spasms, blocked joints, etc. The manufacturer assumed no liability for misuse.

### Asynchronous

Motors A and B can be switched on synchronously or asynchronously.

**Synchronously:** Motors A and B run a synchronous movement for Extension/Flexion and Pronation/Supination.

**Asynchronously:** Both motors run independent of each other in the set range of motion

## CAUTION!

Generally, only **synchronous** operation of both motor is recommended. Asynchronous operation may be medically/therapeutically indicated.

## CAUTION!

Asynchronous motion requires special care and attention of the doctor or assistant in order to prevent the patient from dangers

### Display lighting

This special function sets the lighting of the display field.

Press **SET** to switch the display lighting ON or OFF.

### Language

With choosing the menu item “Language”, the language shown on the LCD display can be changed.

1. Select the function “language” using the +/- keys
2. Press **SET** to activate the language selection. Use the +/- keys to set the desired language
3. Press **SET** to activate the desired language

### Service menu

Only for service matters. Please have a look at the service manual for further pieces of information.

### Saving data

1. To save programmed special functions, press **STOP**
2. To start device, press **START**

## CAUTION!

Before treatment, always perform a test run without and then with the patient.

### Programming example

#### Extension with Pronation/Flexion with Supination

First switch into the programming mode (ch. 5.1). “EXT” will appear in the display.

Select the desired value with the +/- keys.

Now set the Pronation by pressing the **Supination/Pronation** key. “PRO” will appear in the display.

To set the Supination, press the **Supination/Pronation** key again. “SUP” will appear in the display. Use the +/- keys to set up the Supination.

Finally, the Flexion has to be set. To do so, press the **Extension/Flexion** key twice. "FLEX" will appear in the display. Select the desired Flexion value with the +/- keys.

Save your settings by pressing **STOP**.

Once the motion range has been programmed, other program options such as pause, speed, etc. can be used.

## 6. Conversion

The ARTROMOT®-E2 compact can be easily converted, as follows:

1. Set motor A to 30° and motor B to 0° ("New Patient", see special functions, ch.5.3)

2. Unlock the roller brakes and move the ARTROMOT®-E2 compact CPM device to the opposite side of the patient's chair


3. Pull the button (5) down, and swing the upper arm support 180° inwardly. Caution, let the button go by swinging and wait until the "clik", it is locked (see conversion scheme 1a)

4. Loosen the screw (6) and swing the lower arm element 180° inwardly. Tighten the screw (6) again (see conversion scheme 1b)


5. Loosen the screw (7) with 2 rotations and swing the support for the wrist at 180°. Adjust it in the central position again and tighten the screw (7) again (see conversion scheme 2c)

### Conversion scheme ARTROMOT®-E2 compact

①

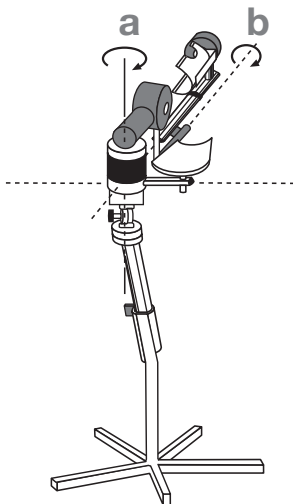②
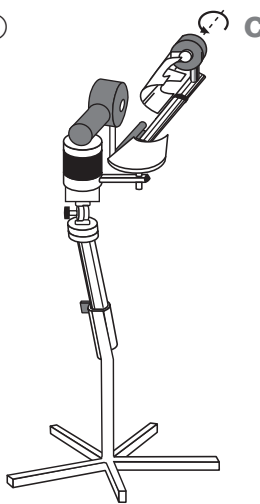
## 7. Shipment, set up and transport

To set up **ARTROMOT®-E2 compact**, please work in the reverse order.

Before using the device, make sure that it is at room temperature. If the temperature during transport was below 0°C (32°F), let the device rest for 2 hours at room temperature to prevent condensation. The device may only be operated in dry rooms.

Preparations for shipment of **ARTROMOT®-E2 compact** by using the original box:

1. Set the motor A to -5° and motor B to 0°
2. Switch off the device and wait until the display expires
3. Pull off the power cord, connectors (14) for the mobility element and the handheld programming unit (24).
4. Remove the upper arm support by pulling the adjustment button at its underside.
5. Swing the tube for the upper arm support to the packaging setting (tube is positioned parallel to Motor A)
6. Loosen the screw for the height adjustment (1) and remove the complete mobility element with motors A and B.
7. Remove the mobility element from the oval tube by loosen the screw (black, round label n°3) and put back the screw into the oval tube.
8. Remove the electronic box with support frame from the star shaped stator support by loosen the screw on the underside and middle of the star shaped stator.
9. Remove the five support legs of the star shaped stator with the provided tool.

To set up the **ARTROMOT®-E2 compact** CPM device, please work in the reverse order.

Before using the device, make sure that it is at room temperature. If the temperature during transport was below 0°C (32°F),

let the device rest for 2 hours at room temperature to prevent condensation. The device may only be operated in dry rooms.

### CAUTION!

**For transport of ARTROMOT®-E2 compact without box please follow the guideline below:**

**Scale 2 = 0°      Scale 3 = 90°**  
**Scale 4 = 0°      Scale 5 = 0°**

Please have a look at unpacking and packing instructions of **ARTROMOT®-E2 compact** for detailed pieces of information and pictures.

Transporting **ARTROMOT®E2 compact**

## 8. Care

### CAUTION!

**The device should always be unplugged before cleaning.**

- The **ARTROMOT®-E2 compact** can be wiped clean with disinfectant and thus fulfills the hygienic requirements for medical equipment
- The armrests can be cleaned with common disinfectants and mild household cleaners
- Only wipe the **ARTROMOT®-E2 compact** CPM device itself with a moist cloth

## **CAUTION!**

**No fluids may be allowed to enter the casing or the hand-held programming unit.**

- Protect the motion device from intensive UV light (sunlight) and naked flames
- After use, the product must be disposed off in accordance with the local legal regulation

## 9. Specifications

Input	100–240 V AC 50/60 Hz
Rated motor current	2 A max
Power consumption	33 VA
Fuses	2 x 1 AT
Degree of protection Class	IPX0 I
Applied part	Type B

Transport dimensions

### **ARTROMOT® E2 compact:**

Length	87.5 cm
Width	57.5 cm
Height	29.0 cm

Adjustment ranges (min/max):

- Height adjustment 31–71 cm  
(measured from seat)
- Lower arm length 29–46 cm

Weight **ARTROMOT®-E2 compact**

17.0 kg

MPG	Class 2a
CSA	IEC 60601-1 + A1: 1993 + A2: 1995 CSA 601.1-M 90 UL 60601-1, Ed.1

## Ambient conditions

Ambient temperature	-24°C to +60°C
Relative air humidity	20% to 85% without condensation
Air pressure	700 hPa to 1060 hPa

## Operational conditions

Ambient temperature	+10°C to +40°C
Relative air humidity	30% to 75%, 90% without condensation
Air pressure	700 hPa to 1060 hPa

---

Subject to change without notice (06/05).

## IEC 601-2:2001

The **ARTROMOT®-E2 compact CPM device** needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided in the accompanying documents.

Portable and mobile RF communications equipment can affect the **ARTROMOT®-E2 compact CPM device**.

The **ARTROMOT®-E2 compact CPM device** should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the **ARTROMOT®-E2 compact** should be observed to verify normal operation in the configuration in which it will be used.

### 1. Electromagnetic emissions

*Guidance and manufacturer's declaration – electromagnetic emissions*

The **ARTROMOT®-E2 compact** is intended for use in the electromagnetic environment specified below. The customer or the user of the **ARTROMOT®-E2 compact** should assure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The <b>ARTROMOT®-E2 compact</b> uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The <b>ARTROMOT®-E2 compact</b> is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not applicable	

## 2. Electromagnetic immunity

*Guidance and manufacturer's declaration – electromagnetic immunity*

The **ARTROMOT®-E2 compact** is intended for use in the electromagnetic environment specified below. The customer or the user of the **ARTROMOT®-E2 compact** should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact  ± 8 kV air	± 6 kV contact  ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-5	± 2 kV for power supply lines  ± 1 kV for input / output lines	± 2 kV for power supply lines  ± 1 kV for input / output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV differential mode  ± 2 kV common mode	± 1 kV differential mode  ± 2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	< 5% $U_T$ (> 95% dip in $U_T$ ) for ½ cycle  40% $U_T$ (60% dip in $U_T$ ) for 5 cycles  70% $U_T$ (30% dip in $U_T$ ) for 25 cycles  < 5% $U_T$ (> 95% dip in $U_T$ ) for 5 s	< 5% $U_T$ (> 95% dip in $U_T$ ) for ½ cycle  40% $U_T$ (60% dip in $U_T$ ) for 5 cycles  70% $U_T$ (30% dip in $U_T$ ) for 25 cycles  < 5% $U_T$ (> 95% dip in $U_T$ ) for 5 s	Mains power quality should be that of a typical commercial or hospital environment. If the user of the <b>ARTROMOT®-E2 compact</b> requires continued operation during power mains interruptions, it is recommended that the <b>ARTROMOT®-E2 compact</b> be powered from an uninterruptible power supply or battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment. Portable and mobile RF communications equipment should be used no closer to any part of the <b>ARTROMOT®-E2 compact</b> , including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.

**NOTE:**  $U_T$  is the a.c. mains voltage prior to application of the test level.

Guidance and manufacturer's declaration – electromagnetic immunity

The **ARTROMOT®-E2 compact** is intended for use in the electromagnetic environment specified below. The customer or the user of the **ARTROMOT®-E2 compact** should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
			<b>Recommended separation distance:</b>
Conducted RF IEC 61000-4-6	3 V <sub>rms</sub> 150 kHz to 80 MHz	3 V <sub>rms</sub>	$d = 1.2\sqrt{P}$
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	$d = 1.2\sqrt{P}$ 80 MHz to 800 MHz
			$d = 2.3\sqrt{P}$ 800 MHz to 2,5 GHz
			<p>where <math>P</math> is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and <math>d</math> is the recommended separation distance in metres (m).</p> <p>Field strengths from fixed RF transmitters as determined by an electromagnetic site survey<sup>a</sup>, should be less than the compliance level in each frequency range<sup>b</sup>.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 

**NOTE 1:** At 80 MHz and 800 MHz, the higher frequency range applies.

**NOTE 2:** These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

- a) Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the **ARTROMOT®-E2 compact** is used exceeds the applicable RF compliance level above, the **ARTROMOT®-E2 compact** should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the **ARTROMOT®-E2 compact**.
- b) Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

### 3. Recommended separation distances between portable and mobile RF communications equipment and the ARTROMOT®-E2 compact

The **ARTROMOT®-E2 compact** is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the **ARTROMOT®-E2 compact** can help prevent electromagnetic interferences by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the **ARTROMOT®-E2 compact** as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1.2\sqrt{P}$	80 MHz to 800 MHz $d = 1.2\sqrt{P}$	800 MHz to 2,5 GHz $d = 2.3\sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at maximum output power not listed above, the recommended separation distance  $d$  in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

**NOTE 1:** At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

**NOTE 2:** These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

## 10. Contacts

Please do not hesitate to contact our local dealers or the following addresses for any further questions about our products and services:

- **ORMED international**  
Please contact your local dealer or us directly:
- **Headquarters Germany**  
ORMED GmbH & Co. KG  
Merzhauser Straße 112  
D-79100 Freiburg  
Tel. +49/761/45 66-01  
Fax +49/761/45 66 55-01
- **USA, St. Paul**  
Tel. 1 (800)-3 28-25 36-73 25  
Fax 1-651-4 15 74 14  
e-mail: r.suddendorf@empi.com
- **Internet**  
www.ormed.de  
e-mail: info@ormed.de

### Important!

**Check all of the components at least once a year for damage and loose connections.**

**Authorized experts must exchange damaged or worn out parts for original spare parts immediately.**

### **Maintenance:**

Basically maintenance-free

### **Warranty:**

2 years (mechanical parts)  
2 years (electronics)

### **Vendor:**

ORMED GmbH & Co. KG  
Merzhauser Straße 112  
D-79100 Freiburg  
Germany

## 11. Technical service

### 11.1 Technical hotline

Do you need any technical assistance?

Telephone: +49-180-5-1 ormed de  
                  +49-180-5-1-67 63 33  
Fax:          +49-180-5-3 ormed de  
                  +49-180-5-3-67 63 33

### 11.2 Changing fuses

#### **CAUTION!**

Only experts may exchange fuses according to DIN VDE 0105 or IEC 364 or comparable standards (such as medical technicians, electricians, electro engineers). Pull the mains plug before opening the case!

#### **CAUTION!**

Only use the appropriate fuses  
2 x 1 AT

### 11.3 Shipment

To prevent damage during transport, only use original packaging for shipment. These boxes can be ordered at ORMED.

## 12. Spare parts

The current spare parts list is provided with the service manual.

When ordering spare parts, please always specify:

- Position
- Description
- Article number
- Quantity
- Serial number of the device for which order is intended

#### **CAUTION!**

Only authorized experts may repair our medical devices.  
ORMED GmbH & Co. KG offers such service training.

Please note the surcharges for reduced quantities on individual spare parts.

Item	Description	Article no.	Quantity
1.	Memory chipcard	0.0034.048	<input type="text"/>
2.	Pen for Memory chipcard	0.0031.006	<input type="text"/>

## Declaration of Conformity

Concerning to the regulations of the Council Directive 93/42/EEC, 14.06.1993 for medical devices the company

ORMED GmbH & Co. KG  
Merzhauser Straße 112  
79100 Freiburg – Germany,

declares, with respect to the following medical devices of the product line

**ARTROMOT®** according to annex

fulfill the requirements as set out in Annex II as well as the essential requirements in Annex I of the said directive.

0297

Freiburg, June 16<sup>th</sup>, 2005

Quality Control Manager

### Annex:

ARTROMOT®-E2 compact  
ARTROMOT®-S2 *PRO*  
ARTROMOT®-S3  
ARTROMOT®-S3 comfort  
ARTROMOT®-SP2 1M  
ARTROMOT®-SP2 2M  
ARTROMOT®-K3  
ARTROMOT®-K4

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ORMED GmbH & Co. KG • Merzhauser Straße 112 • D-79100 Freiburg